

## GARDENING

Spring has arrived! Time to get out in the garden and get planting! Working in the garden is fun and therapeutic. Enjoy the colors and smells, getting dirty, and communicating with your AAC user about the garden. Depending on your device, you might consider using a low-tech board to protect your device from dirt and water.


### CORE WORDS

Do	I
Dig	You
Like	My
Look	It
Love	Here
Need	That
Push	Colors
Put	Dirty
Want	Pretty

### MATERIALS

Seeds/Plants	You
Dirt	AAC User
Pots	AAC Device
Other garden tools	

### Model It on the AAC Device

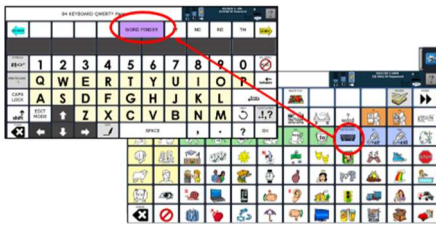

Modeling on your child's device is just "talking" using his/her language! When you say a word with your voice, also say it using your child's device. Look up the words that you are going model using WORD FINDER before you start playing. Start with one word and then add 1-2 words.

Reasons to Communicate	One Word	Two Words	Three (+) Words
Getting Wants/Needs Met (requesting objects/activities/ permission/attention, etc., directing action/to stop, request help)	Want Need That Look Put In Here More Dig	I want Want that Want here Put in Put here In here More here Put more Dig here	I want that I want to dig I need that Need more of that Put that here Put it here Put more here I want to dig Look at that
Exchange Information (share and show objects, confirm/deny, request information, etc.)	I You Look More Here That Dig Push What Color Like	I do I dig Like that You like? I like Put here? What color? Want + color Push here Dig here Look that	I do it You dig here I like that You like it? Put it here? I put it here What color want Push over here Dig in here I want + color Look at that!
Social Closeness/Etiquette (greet, take turns, comment, etc.)	I You More Turn Like Love Cool Pretty Dirty Yuck	I do You do My turn Love it! I like That cool It pretty It dirty You dirty Dirty yuck	This is + color I want to do it I love it! I like that That is pretty. It is so pretty! I love the + color Yuck! Its dirty! It is dirty You got dirty!

### Tips to Engage, Expand, and Succeed:


- Any time you are using water or dirt or anything else that *may* be unsafe for an electronic device, it is safest to use a low-tech form of communication. You can download many low-tech boards from our website here: <https://saltillo.com/chatcorner/content/29> Keep your AAC device safe!
- Gardening can a great time to talk about **colors**! Flowers come in many different colors. Talk about the ones you like, which ones you think are pretty, and which ones you don't like. This is the perfect time to model commenting.
- Some other concepts you could model are:
  - Prepositions: **in, on, under, over**
  - Determiners: **this, that, there, here**
  - Questions: **what, where**
- This could be a great opportunity to work in some STEM (i.e. science) and talk about the plant life cycle using core. In addition to the words above, you can use words like: **grow, big, little, new, young, need(s)**, etc.
- Remember keep your modeling natural and focus on enjoying the task and time. Don't ask too many questions or force responses. Just keep modeling!

Using a low-tech communication board? Or need some choices to help you communicate during this activity?

The following strips can be used as choice boards or fringe vocabulary for the above activity. If you are using this as fringe vocabulary, you will need one of our low-tech flip books or a single page communication board. These boards are available on our website. Download the boards here: <https://aclanguelab.com/resources/unity-manual-communication-boards> You can add these a flip book. OR, you can Velcro these to the top of a single page low-tech core word board.



Attached to a one-page communication board.

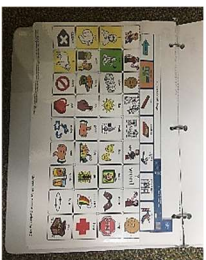

You can add this one to the LAMP WFL, Unity 84, Unity 60 low-tech board.

**gardening Page**

04/18/21 10:13AM  
(6) LAMP WFL 84 Full

You can add me with the Unity 28 low-tech board.

**Garden Page**

04/18/21 10:26AM  
(6) Unity 28 1-FHT