PRC

Active with AAC

DRESS UP

Let's pretend to be someone else. You can be a princess, a police officer, a doctor, an astronaut, or anyone you want! Let your imagination run wild while playing dress up. While you play, there is so much to say and model on your AAC device. Whomever you decide to be, remember to have fun!

CORE WORDS

Give	1
Have	You
Help	It
Like	Me
Look	On
Need	Off
Put	Silly
Try	Funny
Want	Pretty

MATERIALS

Dress up clothes Costumes

AAC Device You Child/client

Model It on the AAC Device



Modeling on your child's device is just "talking" using his/her language! When you say a word with your voice, also say it using your child's device. Look up the words that you are going model using WORD FINDER before you start playing. Start with one word and then add 1-2 words.

Reasons to Communicate	One Word	Two Words	Three (+) Words
Getting Wants/Needs Met	Put	Put on	Put it on
(requesting	Need	Take off	Take it off
objects/activities/	That	Give that	Give me that one
permission/attention,	On	Help me	Try it on
etc., directing action/to	Off	Try on	Look at me
stop, request help)	Give	Help me	Help me please
	Look	You try	You try it on
	Help	Look me	Put it there
Exchange Information	Like	Like that	I like that
(share and show objects,	Have	I have	I have it!
confirm/deny, request	Need	l need	I need that please
information, etc.)	Want	ltry	Try it on
	Who	Try on	Put it on
	Am	Put on	Who is that?
	1	Who that?	Who are you?
	You	Who you?	Take it off
	Try	I like	You want it?
	Put	Take off	Look at this!
	On	You want?	I am XX
Social Closeness/Etiquette	I	l like	I like it
(greet, take turns,	You	l try	You try it
comment, etc.)	My	You try	I try that
	Like	That pretty	Can I try it?
	That	You silly	That is pretty
	Fun	Try it	You look silly
	Pretty	lt fun	I look funny
	Silly	It silly	I feel silly
	Try	Look pretty	This is fun!

Tips to Engage, Expand, and Succeed:

- The most important thing about this activity is to have fun! There are opportunities to make this activity about following directions, putting steps in sequence, and other therapy goals. But, remember it is best to model language in natural settings.
- As previously mentioned, you can take some time to focus on specific language and goals like:
 - $\circ\,$ Following directions: have the child/AAC user put on items of clothing in a specific order, for example. Or give two steps directions.
 - Sequencing: give directions that contain words such first/then (next). Or ask the child/AAC under "what next?" if you are working on putting steps in order.
- After getting dressed in your character, pretend play as your character. This is the perfect chance to keep your language natural! What would your character do and say? Model that!
- You can use this activity as a chance to focus on describing words and commenting. Get dressed in a silly, scary, funny, boring, etc. costume. Comment on how you feel. What does the other person think?

© 2020 PRC-Saltillo. Non-commercial reprint rights for clinical or personal use granted with inclusion of copyright notice. Commercial use prohibited; may not be used for resale. Contact PRC-Saltillo for questions regarding permissible uses.

