

July 2021

## Unity 84 1-Hit Core Word Activity Water Play

Cool off this month with water play. Use a water-safe low tech manual board to keep the conversation going in the pool. Put your words on a kick board, towel, blanket, or laminated poster/manual board.

As the communication partner, whether you are a family member, teacher, therapist or friend, use these words as you talk to model where they are located and how powerful they can be. Look for opportunities to teach and model these words while cooling off with water this month.



Single Words	<b>Pronouns</b>	• I • it • we	<b>Verbs</b>	• play • do • drink • get • go • like	Two Word Phrases	• play more • I do • want play • I want • what do • stop it	• get drink • like more • get out • we play • do it • it good/bad	Three Word Phrases	• we play more • I want play • go play more • get more drink	• what we do • like play it • it good play • we stop play • it bad drink
	<b>Adjectives</b>	• good/bad • more	<b>Question</b>	• what		<b>Bonus: I want get in</b>				



	Hi	What's up?	You?	Fine.	Not good.	Nice meet	I'm going	Take care	Goodbye		
me	my	TALK	can	SOCIAL	that	and	PREPOSITI...	what	a	+s	again
I	we	are	is	were	was	on	to	KEYBOARD	an	the	find
you	they	know	play ...	like ...	work	have	feel ...	read ...	more	fast	STOP
it	he	want ...	FAMILY	come	time	do ...	think	big	get	COLORS	wash ...
she	look	slow	computer	call	good	hear	live	go ...	ride ...	love	help
CLEAR	don't	change	eat ...	make	need	bad	watch ...	drink ...	turn	put	sleep

© 2019 PRC-Saltito. Non-commercial reprint rights for clinical or personal use granted with inclusion of copyright notice. Commercial use prohibited; may not be used for resale. Contact PRC-Saltito for questions regarding permissible uses. (07082020)P/

UNITY® 84 1-Hit for Accent Devices by PRC