

Active with AAC

PLAY-DOH

Let's get outside and play! Kids love to run around, play, and have fun. BUT the playground can also be a great place to model A LOT of language.

CORE WORDS

Can	1		
Cut	lt		
Do	You		
Give	What		
Help	Cool		
Like	Great		
Look	Pretty		
Make	Yummy		
Open	Colors		

MATERIALS

Play-Doh	You
Kitchen toys	Child/Client
Animals	AAC Device
Other toys	
as needed	

Model It on the AAC Device



Modeling on your child's device is just "talking" using his/her language! When you say a word with your voice, also say it using your child's device. Look up the words that you are going model using WORD FINDER before you start playing. Start with one word and then add 1-2 words.

Reasons to Communicate	One Word	Two Words	Three (+) Words
Getting Wants/Needs Met	Open	Open it	Help open it
(requesting	Help	Give it	You open it
objects/activities/	Need	You do	You cut it
permission/attention,	Give	Cut it	Can you help?
etc., directing action/to	More	Want it	Can you do it?
stop, request help)	Do	I want	Make more that
	1	Cut that	Make it big(ger)
	Look	More that	I need more
Exchange Information	Help	Help me	What you make?
(share and show objects,	Look	I help	What is it?
confirm/deny, request	Make/made	Look it	I like it
information, etc.)	Like	I make/made	Look it big
	What	I like	Is it good?
	Feel	What is	I do/did it
	Think	I think	Do you need more?
	Cut	I do/did	I think it good
	Do/did	You cut	It needs more
	Colors (e.g.	What that	I made XX
	red)	Here is	It feels good
Social Closeness/Etiquette	Like	Like that	You did it
(greet, take turns,	Cool	That cool	I like that
comment, etc.)	Fun	I like	That is great
	Great	That yummy	That is cool
	Yummy	My turn	It is yummy
	1	I do	It is pretty
	You	That good	I will do it
	Turn	I love	You do it
	Do	That funny	It's my turn

Tips to Engage, Expand, and Succeed:

- Let your imagination run wild! You can pretend to cook, make animals, make a town, or a race track. There is so much to do with playdoh. If you make foods or animals, you could use the **Groups** page to focus on some of that fringe vocabulary. There are pages for animals, foods, etc.
- Looking for ways to incorporate some basic concepts? You could make **big** and **little** versions of something. Take turns hiding a Play-Doh animal, when you find it you have to say where it is. Is it **on** the shelf? **In** the box? **Out** of the room? **Under** the chair?
- Don't anticipate all the needs of the client. Wait for the client/child to ask for help to
 open the Play-Doh jar. Do not help them make an animal/food/etc. until they ask for
 help.
- Whatever you play, keep it natural. Did you make playdoh food? Pretend to eat.
 Was it good/yummy? Was it hot? Did you make an animal? Where is the animal
 going? Is it a nice or scary animal? Just relax and play without forcing
 communication or asking too many "What's this"-type questions.

https://www.youtube.com/channel/UCT1pgntWkiA5rKlOvZGXoYA Using a PRC device and don't see a word you want to say? Find a quick video on how to add a word to your device:

will need a single page communication board. These boards are available on our website. Download the boards here: The following strips can be used as choice boards or fringe vocabulary for the above activity. If you are using this as fringe vocabulary, you Using a low-tech communication board? Or need some choices to help you communicate during this activity?



https://aaclanguagelab.com/resources/unity-manual-communication-boards You can add these a three ring binder, OR you can Velcro these to the top of a single page low-tech core word board.






- 3 - 3

Attached to a one-page communication board.

11111

ĺ

າ add this one to the LAMP WFL, Unity 84, Unity 60 low tech board

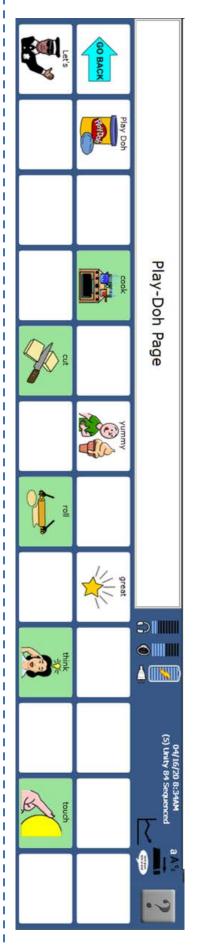

add this one to the Unity 28, 36 or 45 low tech board.

